VIPI CRIL PLUS

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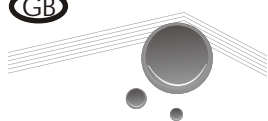
A:07/09

VIPI CRIL PLUS

Heat-curing denture base acrylic resin (powder and liquid), Indicated for Full Removable Prosthesis, Fixed Prosthesis (over denture) and Partial Removable Prosthesis.

Instructions: SHAKE THE POWDER-RECIPIENT BEFORE USE.

•Preparation:

The mixing-recipient, hands and spatulas have to be free of any contaminants to avoid changes in coloration. We recommend an ambient temperature between 15°C and 30°C for mixture and application. Observe the correct dosage for preparation of the VIPI CRIL PLUS dough. Dose 6,5 ml (little cup of the VIPI dosage aid) of the liquid with Crosslink to a preferable glass recipient. Add 14g (big cup of the VIPI dosage aid) of the desired shades of VIPI CRIL PLUS resin. Use a plastic spatula to mix the dough until total homogenization. Take care to avoid the incorporation of air bubbles. Close the mixing recipient and wait for the dough-stage which is reached when the resin dough may be removed in one piece from the mixing recipient (approximately 10 minutes after dosage, at 23°C).

•Packing:

Once the dough stage is reached, the resin shall be packed to the prepared and isolated (VIPI FILM) flask and covered by a high density polyethylene sheet. Never use cellophane !!! Close the flask and execute the first trial packing with a pressure of 750 kgf to 1000 kgf. After the trial closure, open the flask, remove the plastic sheet and cut the resin excess to avoid a vertical dimension increase. Isolate the base model again and close the flask for final packing at 1000 to 1250 kgf. Before polymerization, store the packed resin for at least 20 minutes. For more translucent and porosity-free prosthesis, it is recommended to let the resin rest for about 2 hours.

•Polymerization:

After final packing, close the flask in a polymerization clamp or, in case of using the VIPI STG Microwave Flask, tighten the 4 bolts and take the flask to the polymerization bath. The VIPI CRIL PLUS resin is very versatile and may be polymerized by many cycles of polymerization. Two examples for polymerization cycles to achieve a residual monomer content less than 1 % follow.

•Conventional polymerization:

Put the flask into 3 liters of cold water, switch the heat source (electric or gas) on low level and let bath temperature reach 70°C. Maintain this temperature for 30 minutes. Increase the heat source to reach 100°C and let the bath boil for 90 minutes. Switch off the heat source and let the flask cool inside the bath to 40°C before initiating the deflasking.

•Thermo-pneumatic polymerization:

Put the flask into a thermo-pneumatic polymerization equipment and cover the flask with water. Close the equipment and inject 5 bar (60 psi) pressurized air. Switch the electrical heater on until the temperature reaches 120°C, switch the heat source off and let it cool down to 90°C. Turn on the heat source again until the temperature reaches 120°C. Turn off the heating and let it cool down to 60°C and make sure to depressurize before opening the thermo-pneumatic polymerization equipment and start the deflasking.

•Deflasking:

The denture is deflasked through the normal method when the flask is cold, taking care not to damage the teeth or the denture.

•Surface finish and polish:

Initiate the finish with adequate mills, followed by grinding disks and stones and finally fine grained grinding papers. The polish may be realized with smooth textile brush, pumice and Spanish white.

•Note: The acrylic liquid VIPI CRIL PLUS is sold in two different options: regular or additivated with a crosslinking agent which improves the impermeability properties avoiding impregnations, unpleasant odors, resulting in dentures that do not suffer alterations and irritations.

The use of the Crosslink additivated liquid facilitates the surface finish and ensures a permanent gloss. VIPI recommends for this product the use of mechanical polishing, only.

Precautions:

- Follow usual procedures for infection-control, waxing-up, articulation and maintenance of the prosthesis, as well as for individual protection;
•For dental professional use only;
•Liquid: Non-toxic very inflammable material (Flash Point: 10°C/50°F);
•Powder: May form inflammable resin/ air mixtures;
•For all acrylic resins, handling in ventilated places, using nitrile-gloves and eye-protection, is highly recommended;
•The product should not be handled during pregnancy, lactancy or by known methacrylate sensitive individuals;
•Store the product in a cool place, protected from direct sunlight;
•Do not allow the products to reach the sewage system, nor discard them with the domestic waste.

Composition:

Powder: Liquid with Crosslink:
•Polymethylmethacrylate •Methylmethacrylate
•Benzoyl Peroxide •EDMA (Crosslink)
•Biocompatible Pigments •Inhibitor

Liquid without Crosslink:

•Methylmethacrylate
•Inhibitor

Shades:

Powder:
With veins: Light Pink, Regular Pink, Regular Pink-B, Dark Pink, Special Pink and Black Pink;
Without veins: Clear, Translucent Red and Palato.

Packages:

Powder: Containers of 80g, 225g, 450g, 1000g, 2.250kg, 5kg, 25kg and 50kg.
Liquid: With Crosslink: Bottles of 50ml, 120ml, 250ml, 500ml, 1000ml and 5 liters (Can).
Without Crosslink: Bottles of 50ml, 120ml, 250ml, 500ml, 1000ml and 5 liters (Can).

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Madespa S.A.